NSK

Please read this Operation Manual carefully before use, and file for future reference

Micromotor with LED

NLX nano / NLX plus

MADE IN JAPAN

OM-E0498E 001

Powerful Partners®

CAUTION

- When operating this motor always consider the safety of the patient.
- The motor is designed only for clinical dental use by qualified personnel.
- This motor can not operate without exclusive control unit.
- Do not allow any impact on to the motor. Do not drop it. It is cause a malfunction.
- Check for vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the motor immediately and contact authorized dealer.
- Do not exceed the drive motor speed recommended the handpiece (Option) manufacturers.
- Prior to use, always check the handpiece tightness, vibration, noise and overheatin. If any abnormalities are detected, stop using immediately and contact authorized dealer.
- Do not use contaminated air (by dust or moisture). If the air contains water or dust, it might cause a malfunction or overheat.
- Do not use this motor under strong stress for long time. It is cause overheat.
- Do not connect or disconnect the cord until the drive motor has completely stopped.
- Do not connect / disconnect the handpiece during operation.

- Verify that the Speed Control Switch is adjusted within the allowable speed before use.
- Air Requirements: dry, free from contamination and oil. Use a compressor with a dry air system. Install an air filter if necessary. Blow out the lines before installation.
- Do not Autoclave (or any other high temperature Sterilization) Motor Cord.
- The user shall be responsible for operation and maintenance.
- The operator is responsible for correct operational control, maintenance and inspection.
- Surface temperature shall be more 60°C without using a coolant air, to avoid this event, be sure to use coolant air.

1. Specification

Model	NLX nano	NLX plus
Max. Rated input	Electric voltage AC19V	Electric voltage AC25V
	Electric current 5A	Electric current 5A
Max. Rotation Speed	1,000 - 40,000 min ⁻¹	100 - 40,000 min ⁻¹
Max. Torque	3.4 N·cm	4.0 N·cm
Dimensions	DØ22 x H70 mm	DØ22 x H77 mm

Spray Supplying water	More than 65 mL/min (0.2MPa)
Chip Air	More than 1.5 L/min (0.2MPa)
Coolant Air	More than 6.5 NL/min (0.2MPa)

2. Motor

2-1 Connecting / Disconnecting the Motor and the Motor Cord

Align and insert firmly the motor pin into the pin holes of the Motor Cord Connector, and fasten the Motor Cord Nut securely.(Fig. 1)

To remove the Motor Cord from the Motor, unscrew and detach the Motor Cord Nut, and gently pull out the Motor Cord Connector.

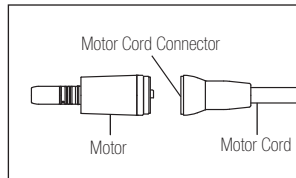
Fig. 1

2-2 Connectioning / Disconnecting the motor and the handpiece (Option)

To insert the E-type handpiece in the Motor Insert, align handpiece and Motor, then turn those until its click. (Positioning pins are aligned)

To remove the handpiece, simply pull out the handpiece from the Motor.

*Handpiece is not included.

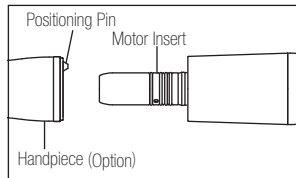
Fig. 2

CAUTION

- Care should be taken when using any handpiece attachment which requires lubrication (E Type Micromotor). Following lubrication stand the handpiece attachment vertically until the excess solution drains. Wipe the handpiece then attach it to the Micromotor.
- Do not connect or disconnect the handpiece until the motor has completely stopped.

3. Sterilization

For the sterilization method, we recommend the autoclave sterilization method. Sterilization is required first time you use and after each patient as noted below.

■ Autoclaving

- 1) Turn off the Power.
- 2) Detach the motor from the Motor Cord. (Refer to 2-1 Connecting/disconnecting the Motor and the Motor Cord)
- 3) Clean the surface of the Motor with brush. (Do not use the metal brush), and wipe it with the cotton moistened with disinfecting alcohol.
- 4) Screw the Motor Cap to the Motor. Put the Autoclave plug to the motor insert.
- 5) Insert into an autoclave pouch. Seal the pouch.
- 6) Autoclavable up to max. 135°C.
ex.) Autoclave for 20 min. at 121°C, or 15 min. at 132°C.
- 7) Keep the Motor in the autoclave pouch to keep it clean until you use it.
* Sterilization at 121°C for more than 15 minutes is recommended by EN13060.

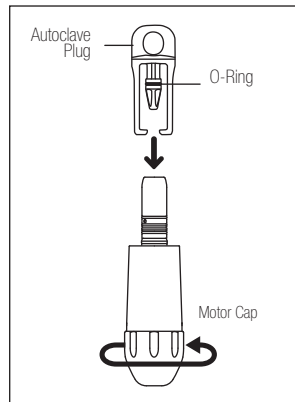
Fig. 3

 **CAUTION**

- Do not Autoclave (or any other high temperature Sterilization) Motor Cord.
- Do not lubricate the motor.
- Do not autoclave the motor, even when it is bagged, together with other instruments, from which chemicals are not sufficiently cleaned. This is to avoid discoloration and/or damage to the plating of the motor.
- Do not wash, soak, or wipe off the motor with/in the oxidation potential solution (strong acid, super acid solution) sterilized solution, strong alkaline solution or functional water.
- The motor is kept in suitable atmospheric pressure, temperature, humidity and ventilation. Moreover, air does not include a dust, salt and sulfur.
- Do not sterilize with dirt on the surface. It might cause rust.
- You can hang Autoclave Plug on the Hunger Post to prevent loosing it.
- Do not hung the Autoclaving Plug with motor. It may drop the motor and cause damage.
- Do not touch the handpiece just after the autoclave. It may very hot.
- We do not recommend you to sterilize on plasma sterilization or EOG stabilization.

4. Maintenance

If the O-Ring became depleted, such as difficult to connect the handpiece or air or water might leak, replace the O-Ring.

Remove the O-Ring at the Motor Insert with a pointed tool, and mount the new O-Rings into the groove.

O-Ring (Blue) : 0313084070

O-Ring (Black) : 0312074080

O-Ring (Autoclave Plug): 0312457102


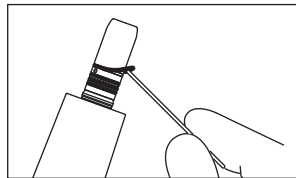
Fig. 4

CAUTION

- There are 4 O-Rings on the Motor Insert. Blue one is thinner than other 3 pieces. Make sure to order it.
- Do not use Autoclave Plug without the mounting O-Ring. It may cause a malfunction. If O-Ring had damaged, replace it immediately.
- It might happen if the O-Ring becomes depleted.
 - Air/Water leak
 - Air/Water does not come off
 - Vibration
 - Difficult to connect/disconnect the handpiece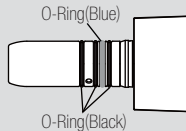

5. Troubleshooting

If the motor stops due to an abnormality such as a malfunction, overload, break or incorrect usage, it automatically checks the state of the Control Box and detects the cause of the abnormality and displays an error code on the Speed Indicator. If an error code is displayed, turn on the power again and check whether the same error code is displayed. If the same error code is displayed, take action by referring to the instructions provided in the "Check/Remedy" column in the following table.

Check/Remedy	Cause	Solution
Motor does not run	Motor Cord is not connected correctly	Check the connection
LED does not light	Reached the end of life expectancy	Contact your dealer
Water leakage	Motor Cord is not connected correctly	Check the connection
	Water is leaking from the motor	Replace with new O-Ring

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NSK France SAS
NSK Oceania Pty Ltd
NSK Asia Pte Ltd