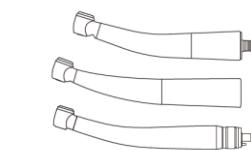
This document is an excerpted version of the operation manual (OM-T0517E002) which contains only a single language.

Ti-Max Z Pana-Max2
Ti-Max X450 Pana-Max PLUS
S-Max M Pana-Max PLUS2
S-Max pico FX Plus

Download operation manuals

www.nsk-dental.com/support/userguide.html

NAKANISHI INC. www.nsk-inc.com 700 Shimohinata, Kanuma, Tochigi 322-8666, Japan
NSK Europe GmbH Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

User: Qualified professionals (dentist)

Patient population : Any patient (except babies). The user's discretion is required prior to use.

Indications for use:

This product is pneumatic dental drilling system handpiece intended for removing carious material, reducing hard tooth structure, cavity preparation, finishing tooth preparations and restorations and polishing teeth.

Only for Ti-Max X450 series

The Ti-Max X450 series is pneumatic dental drilling system handpiece with intended use of being a surgical tool for impacted third molar removal and periodontal procedures for which a conventional handpiece would be used.

GS1 DataMatrix for Unique Device Identifier.

Prohibited / Process NOT permitted / Wrong

Do the step / Process permitted / Correct

If.....(Hypothetical situation)

Call for maintenance

Recommend

Do not use any object with a sharp edge.

Drinkable tap water

Move in the direction indicated.

Move fully to the stop, in the direction indicated.

Blowing off with compressed air

Specifications

Drive air pressure

Chip air pressure

Water Pressure

Exhaust air pressure

Caution, hot surface

State of being fixed.

Drying time

The instrument can be used with a washer-disinfector for thermal disinfection.

WARNING WARNING: Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.

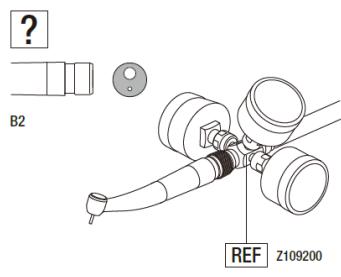
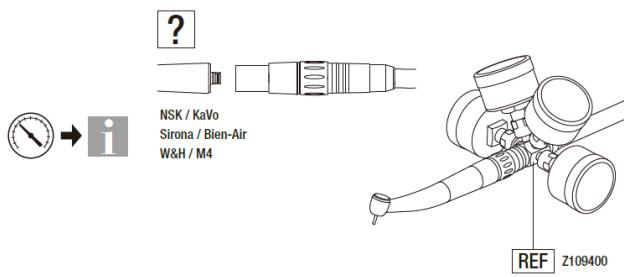
CAUTION CAUTION: Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

Depressing the Push Button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue.







- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not disassemble or alter the handpiece and cartridge, etc. except as recommended by NSK in this Operation Manual.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- The product is delivered in a non-sterile condition. Be sure to sterilize the product and the bur prior to use.
- Perform regular function and maintenance checks.
- Avoid continual eye contact with the LED light. (Z800BL / Z800WL / Z900BL / Z900WL / X450BLED / X450WLED / M800BL / M500BLED / M800WL / M500WLED / M900BL / M600BLED / M900WL / M600WLED / S-Max pico BLED / S-Max pico WLED)
- This handpiece is a LED product classified in the "Exempt Group" (no photobiological hazard) in accordance with IEC62471/EN62471.
- Use a power source which meets the following requirements.
 1. The electricity supply of the power source is below 15W both under normal and single-failure conditions.
 2. The power source uses a SELV circuit for electricity supply.
 3. The output voltage of the power source is within the range recommended by the manufacturer of this product.
- Contact your Authorized NSK Dealer if repairs are necessary.
- An accidental symptom, such as subcutaneous emphysema may be developed in the use of this product for extraction of wisdom teeth. Do not direct air exhausted from the handpiece to the bleeding area (wound) or into periodontal pockets.
- Report all serious events related to the product immediately to the manufacturer and the responsible national authority.









WARNING Follow the specifications described in this operation manual as to supply pressure, the bur to be used, etc.

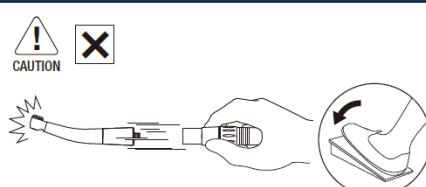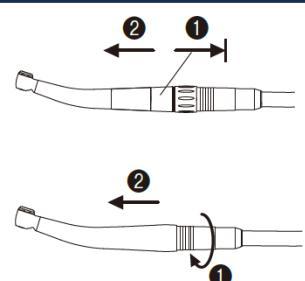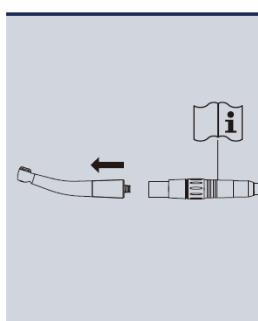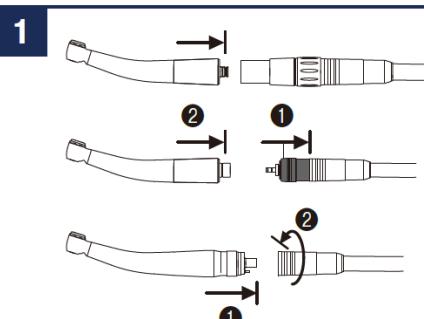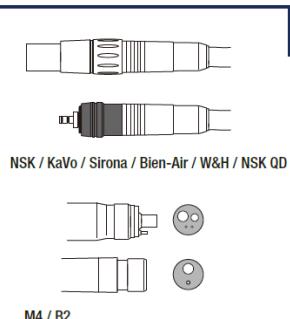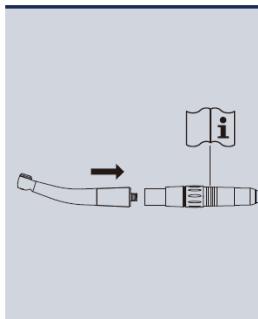CAUTION Do not use air contaminated by dust, moisture or oil.

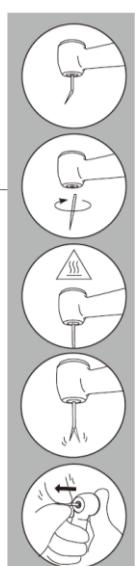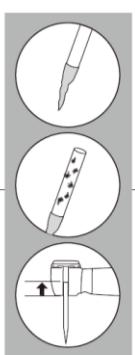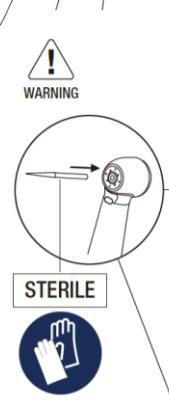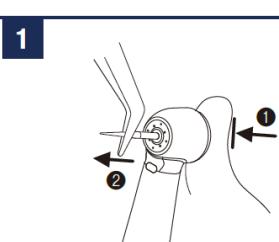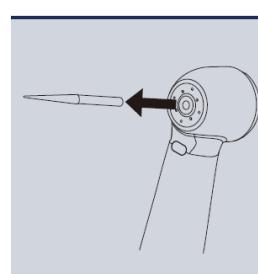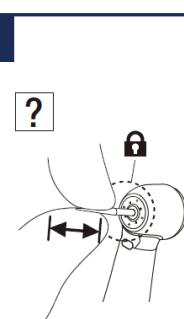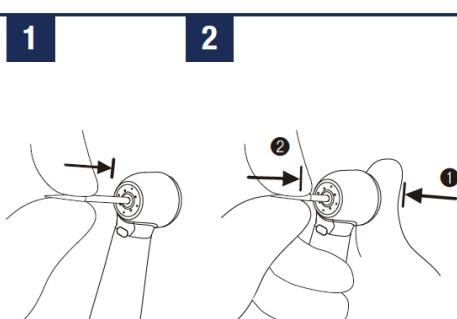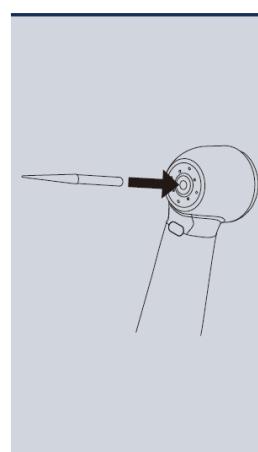Sudden release of the handpiece may cause injury.


WARNING If blood or debris is stuck, wipe in and around the bur insertion hole with a cloth moistened with disinfectant solution until all residues are thoroughly removed. Residual blood or debris may enter the handpiece while changing burs, which could lead to higher risk of cross-contamination.

CAUTION Always follow the instructions provided by the bur manufacturer regarding the bur usage and rotation speed, etc.

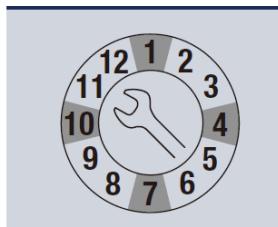CAUTION Do not apply excess pressure to the bur as it may break or bend and become difficult to remove.





1 | 2

1 | 2 | 3

Checking the Handpiece Before
Each Use

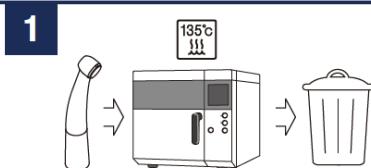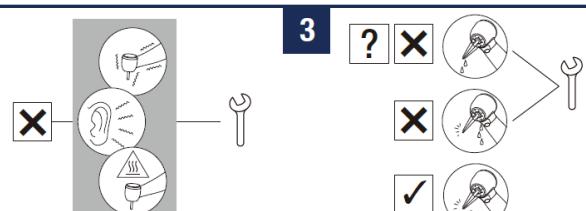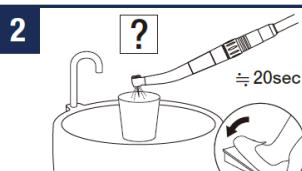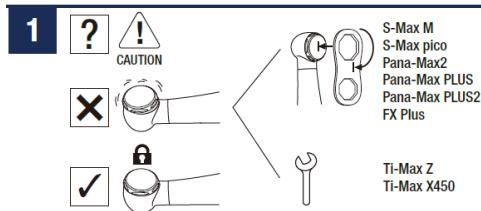Periodical Maintenance Checks
Every 3 months

Be sure to sterilize before disposing. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose the product for you.

EN Warranty	SK Záruka
DE Garantie	HU Jótállás
FR Garantie	SL Garancija
ES Garantía	HR Jamstvo
IT Garanzia	SV Garanti
PT Garantia	DA Garanti
RU Гарантия	ZH 保固
RO Garanție	NL Garantie
CS Záruka	PL Gwarancja
BG Гаранция	TR Garanti
EL Εγγύηση	FI Takuu

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

WARNING To ensure cleaning and sterilization efficacy, use only the following procedures for reprocessing.

WARNING Follow local rules, regulations, and guidelines regarding the reprocessing of devices.

WARNING Immediately after a treatment (within 1 hour), perform maintenance and then store the handpiece. Failure to properly maintain the handpiece may cause infection, product failure, overheating leading to burn injuries.

CAUTION Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong

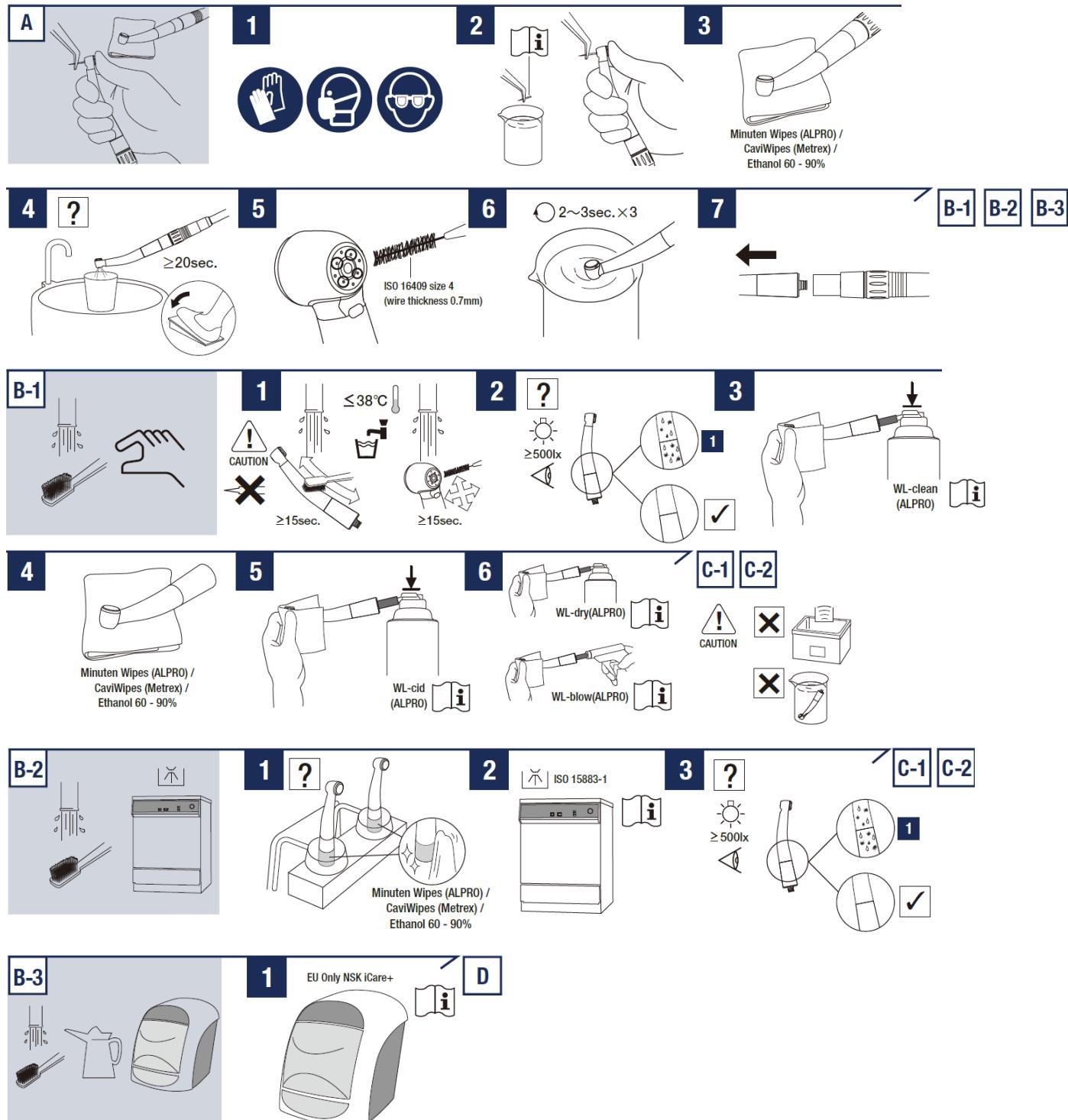
acid/alkaline chemicals, chlorinecontaining solutions, solvents such as benzine or thinner.

WARNING If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components and may lead to blood coagulation inside the handpiece. Coagulated blood may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately lubricate those handpieces with PANA SPRAY Plus.

CAUTION Store the product in a well ventilated place out of direct sunlight and within the range of temperature, humidity and pressure specified in specifications.


CAUTION Remove the handpiece from the washer-disinfector immediately after the cleaning, disinfecting, and drying cycle

is complete (within 1 hour), otherwise it may cause corrosion.

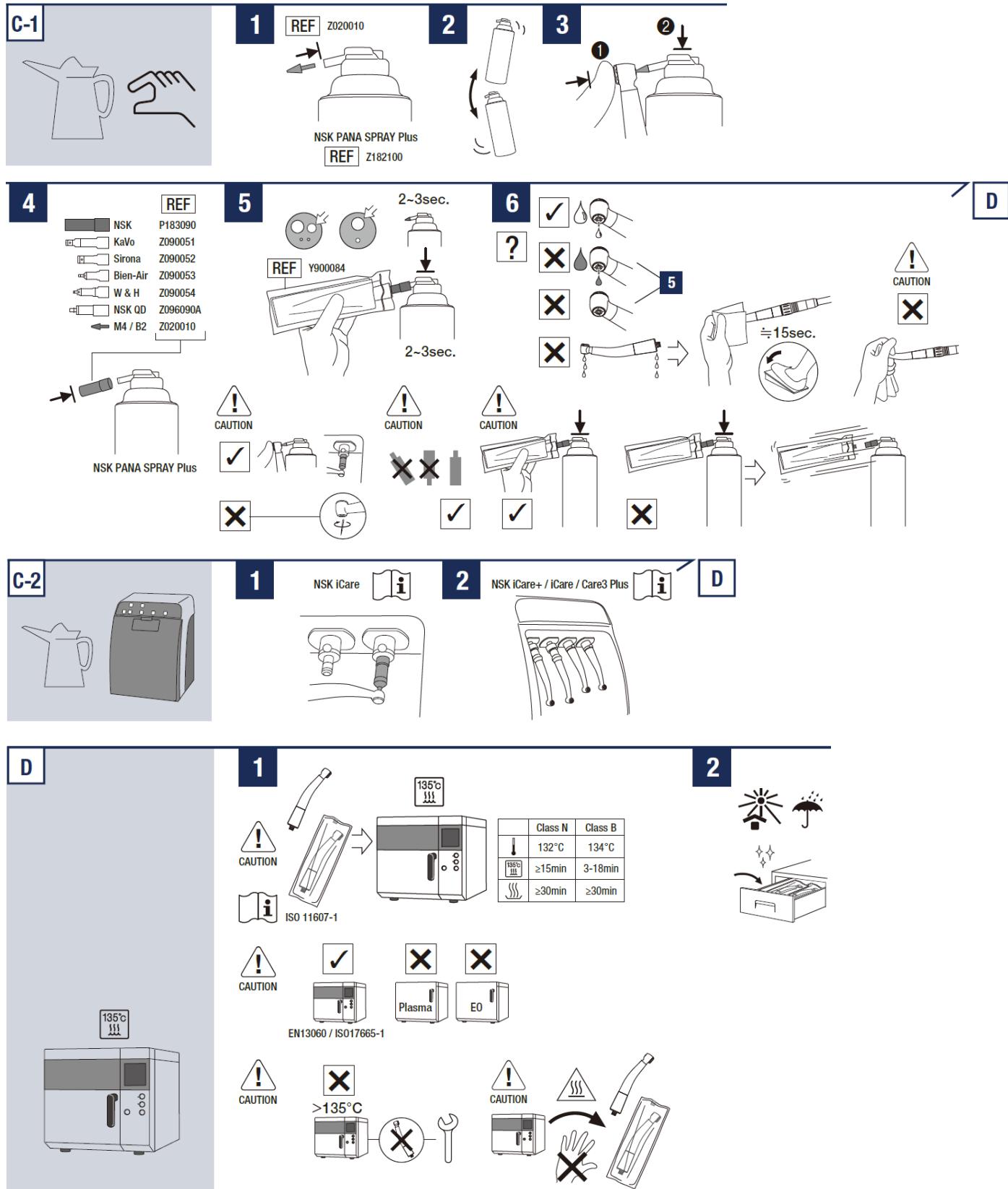CAUTION After washing with washer-disinfector, prior to lubrication, dry the product until all internal moisture is completely removed. Remaining moisture inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

CAUTION It is verified that this product can be sterilized at least 250 times.

CAUTION Clean and lubricate the handpiece prior to sterilization. If blood remains on the internal surface it may become clotted and cause product failure.

CAUTION Immediately after sterilization is complete (within 1 hour), remove the handpiece from the sterilizer. Failure to do so may cause corrosion.

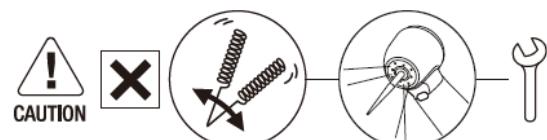
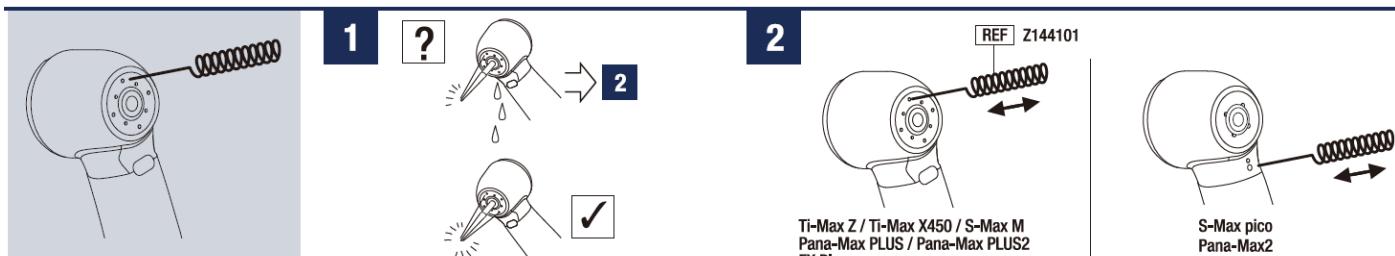
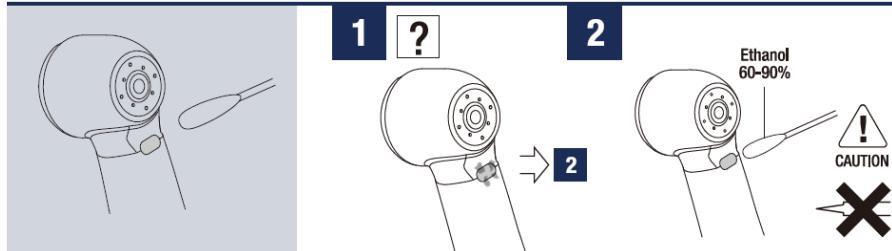
CAUTION Do not perform steam sterilization on the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.

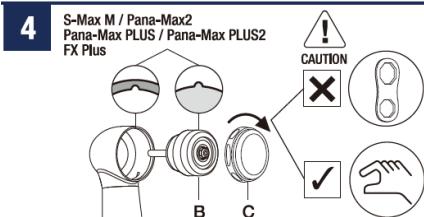
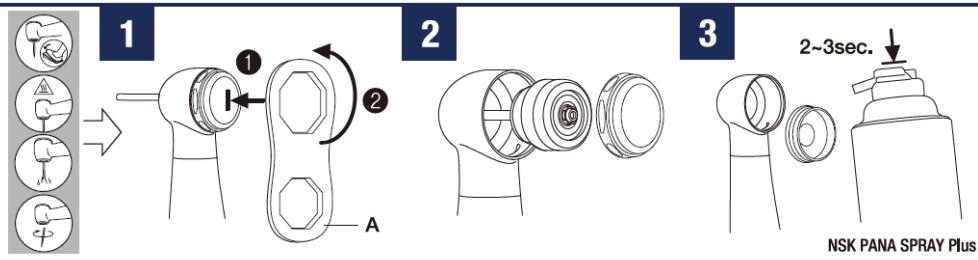
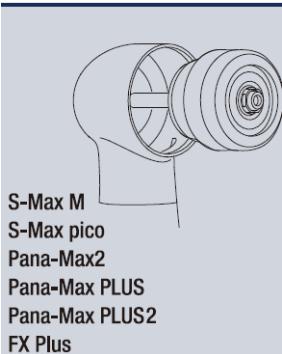
CAUTION Sterilization is not guaranteed after the sterilization retention period specified by the manufacturer and seller of the sterilization pouch has elapsed. If the sterilization retention period has elapsed, perform sterilization again with a new sterilization pouch.



CAUTION Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.



	A	B	C
S-Max M800 / M500 Pana-Max PLUS MU Pana-Max PLUS2 MU FX Plus MU	20000535	P1280	P1001500
S-Max M900 / M600 Pana-Max PLUS SU Pana-Max PLUS2 SU FX Plus SU	20000535	P1281	P1027500
S-Max pico	20002803	P1140200	P1140500
Pana-Max2	P1190765	P1190	P1190050






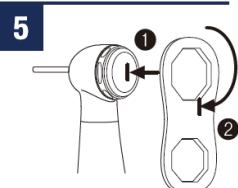
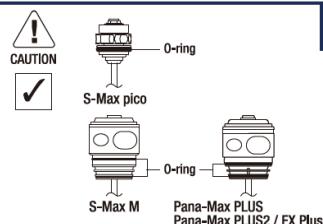
If another cartridge is used NSK cannot guarantee performance and the handpiece warranty would become invalid.

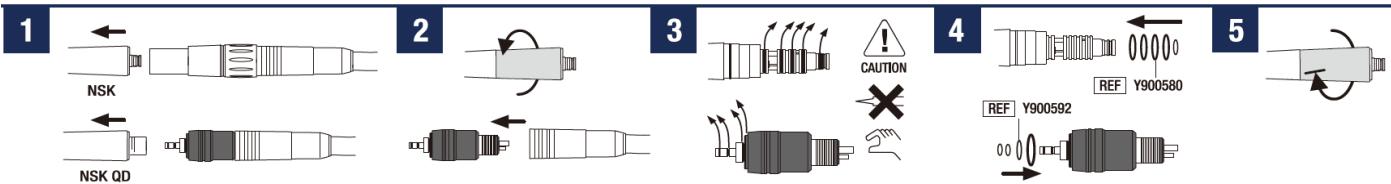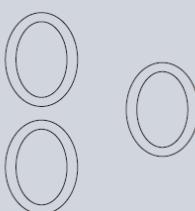Replace the O-rings if water is present in the exhaust air line. This is an indication of possible water leakage within the coupling. ALWAYS change the complete set of O-rings.

CAUTION Do not force the new replacement O-ring with excessive pressure.

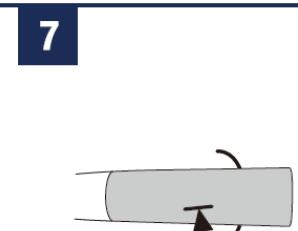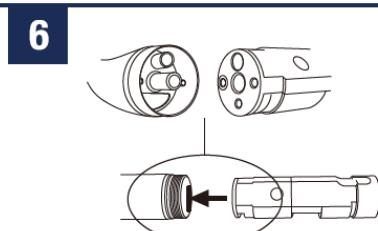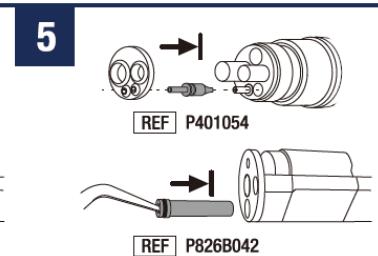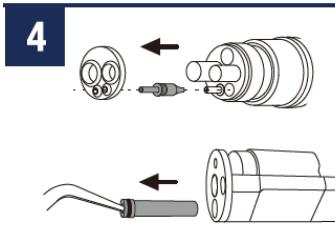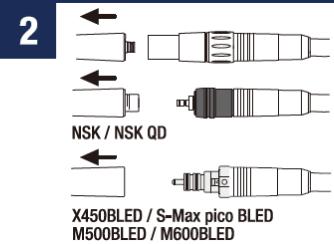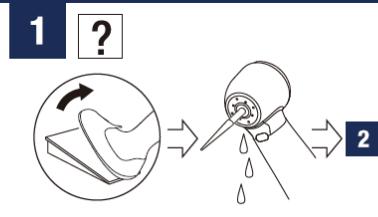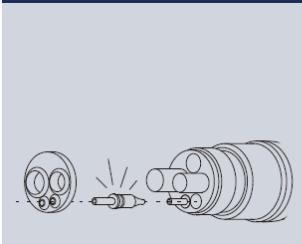CAUTION When inserting new O-rings, make sure they are inserted in the correct groove.

CAUTION Make certain that the taper ring is firmly tightened. If the taper ring is loose water and air leakage could occur.

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Model		Model	
Z800L		Z800L	✓ ✓ ✓
Z800	-	Z800	✓ ✓ ✓
Z800KL	Glass Rod	Z800KL	✓ ✓ ✓
Z800K	-	Z800K	✓ ✓ ✓
Z800SL	Glass Rod	Z800SL	✓ ✓ ✓
Z800BL	-	Z800BL	✓ ✓ ✓
Z800WL	Glass Rod	Z800WL	✓ ✓ ✓
Z800W	-	Z800W	✓ ✓ ✓
Z900L	Glass Rod	Z900L	✓ ✓ ✓
Z900	-	Z900	✓ ✓ ✓
Z900KL	Glass Rod	Z900KL	✓ ✓ ✓
Z900K	-	Z900K	✓ ✓ ✓
Z900SL	Glass Rod	Z900SL	✓ ✓ ✓
Z900BL	-	Z900BL	✓ ✓ ✓
Z900WL	Glass Rod	Z900WL	✓ ✓ ✓
Z900W	-	Z900W	✓ ✓ ✓
X450L	Glass Rod	X450L	✓ ✓ ✓
X450	-	X450	✓ ✓ ✓
X450KL	Glass Rod	X450KL	✓ ✓ ✓
X450SL	-	X450SL	✓ ✓ ✓
X450LED	White LED	X450LED	✓ ✓ ✓
X450WLED	AC/DC 3.3±0.05V	X450WLED	✓ ✓ ✓
X450M4	-	X450M4	✓ ✓ ✓
X450D	-	X450D	✓ ✓ ✓
M800L / M500L	Glass Rod	M800L / M500L	✓ ✓ ✓ ✓
M800 / M500	-	M800 / M500	✓ ✓ ✓ ✓
M800KL / M500KL	Glass Rod	M800KL / M500KL	✓ ✓ ✓ ✓
M500K	-	M500K	✓ ✓ ✓ ✓
M800SL / M500SL	Glass Rod	M800SL / M500SL	✓ ✓ ✓ ✓
M800WL / M500WLED	-	M800WL / M500WLED	✓ ✓ ✓ ✓
M800BL / M500BLED	White LED	M800BL / M500BLED	✓ ✓ ✓ ✓
M900L / M600L	Glass Rod	M900L / M600L	✓ ✓ ✓ ✓
M900 / M600	-	M900 / M600	✓ ✓ ✓ ✓
M900KL / M600KL	Glass Rod	M900KL / M600KL	✓ ✓ ✓ ✓
M900SL / M600SL	-	M900SL / M600SL	✓ ✓ ✓ ✓
M900K / M600K	-	M900K / M600K	✓ ✓ ✓ ✓
M900WL / M600WLED	White LED	M900WL / M600WLED	✓ ✓ ✓ ✓
M900BL / M600BLED	AC/DC 3.3±0.05V	M900BL / M600BLED	✓ ✓ ✓ ✓
M900W / M600W	-	M900W / M600W	✓ ✓ ✓ ✓
S-Max pico	Glass Rod	S-Max pico	✓ ✓ ✓ ✓
S-Max pico KL	-	S-Max pico KL	✓ ✓ ✓ ✓
S-Max pico SL	White LED	S-Max pico SL	✓ ✓ ✓ ✓
S-Max pico BLED	AC/DC 3.3±0.05 V	S-Max pico BLED	✓ ✓ ✓ ✓
S-Max pico WLED	-	S-Max pico WLED	✓ ✓ ✓ ✓
S-Max pico M4	≥1,5NL/min (0,20MPa)	S-Max pico M4	✓ ✓ ✓ ✓
S-Max pico B2	-	S-Max pico B2	✓ ✓ ✓ ✓
S-Max pico QD	≥1,5NL/min (0,20MPa)	S-Max pico QD	✓ ✓ ✓ ✓
Pana-Max2 M4	≥1,5NL/min (0,20MPa)	Pana-Max2 M4	✓ ✓ ✓ ✓
Pana-Max2 B2	-	Pana-Max2 B2	✓ ✓ ✓ ✓
Pana-Max2 QD	-	Pana-Max2 QD	✓ ✓ ✓ ✓
Pana-Max2 PTL	≥1,5NL/min (0,20MPa)	Pana-Max2 PTL	✓ ✓ ✓ ✓
Pana-Max2 KV	-	Pana-Max2 KV	✓ ✓ ✓ ✓
PAP-MU M4 / PAP2-MU M4 / FXP-MU M4	≥1,5NL/min (0,20MPa)	PAP-MU M4 / PAP2-MU M4 / FXP-MU M4	✓ ✓ ✓
PAP-MU B2 / PAP2-MU B2 / FXP-MU B2	-	PAP-MU B2 / PAP2-MU B2 / FXP-MU B2	✓ ✓ ✓
PAP-QD MU / PAP2-MU QD / FXP-MU QD	≥1,5NL/min (0,20MPa)	PAP-QD MU / PAP2-MU QD / FXP-MU QD	✓ ✓ ✓
PAP-SU M4 / PAP2-SU M4 / FXP-SU M4	-	PAP-SU M4 / PAP2-SU M4 / FXP-SU M4	✓ ✓ ✓
PAP-SU B2 / PAP2-SU B2 / FXP-SU B2	-	PAP-SU B2 / PAP2-SU B2 / FXP-SU B2	✓ ✓ ✓
PAP-QD SU / PAP2-SU QD / FXP-SU QD	≥1,5NL/min (0,20MPa)	PAP-QD SU / PAP2-SU QD / FXP-SU QD	✓ ✓ ✓