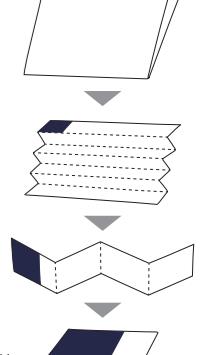
www.nsk-dental.com/support/userguide.html

NANKANISHI INC. ■ www.nsk-dental.com NSK Europe GmbH ■

700 Shiminohata, Kanuma, Tochigi 322-8866, Japan Elly-Beimann-Strasse 8, 65760 Eschborn, Germany

## OPERATION MANUAL

CE MADE IN JAPAN

EN Patient population - Any patient except babies. The user's discretion is required prior to use.

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EN Preferred population - Any patient except babies. The user's discretion is required prior to use.

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EN Indications for use: This product is a pneumatic dental handpiece intended for use in dentistry for the preparation, finishing, tooth structure, cavity preparation, finishing tooth preparations and restorative procedures.

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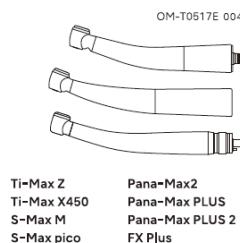
This document is an excerpted version of the operation manual (OM-T0517E004) which contains only a single language.

Download operation manuals

[www.nsk-dental.com/support/userguide.html](http://www.nsk-dental.com/support/userguide.html)

NAKANISHI INC. www.nsk-dental.com  
700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK Europe GmbH

Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

User: Qualified professionals (dentist)

Patient population : Any patient (except babies). The user's discretion is required prior to use.

Indications for use:

This product is pneumatic dental drilling system handpiece intended for removing carious material, reducing hard tooth structure, cavity preparation, finishing tooth preparations and restorations and polishing teeth.

Only for Ti-Max X450 series

The Ti-Max X450 series is pneumatic dental drilling system handpiece with intended use of being a surgical tool for impacted third molar removal and periodontal procedures for which a conventional handpiece would be used.

GS1 DataMatrix for Unique Device Identifier.

Prohibited / Process NOT permitted / Wrong

Do the step / Process permitted / Correct

If.....(Hypothetical situation)

Call for maintenance

Recommend

Do not use any object with a sharp edge.

Drinkable tap water

Move in the direction indicated.

Move fully to the stop, in the direction indicated.

Blowing off with compressed air

Specifications

Drive air pressure

Chip air pressure

Water Pressure

Exhaust air pressure

Caution, hot surface

State of being fixed.

The instrument can be used with a washer-disinfector for thermal disinfection.

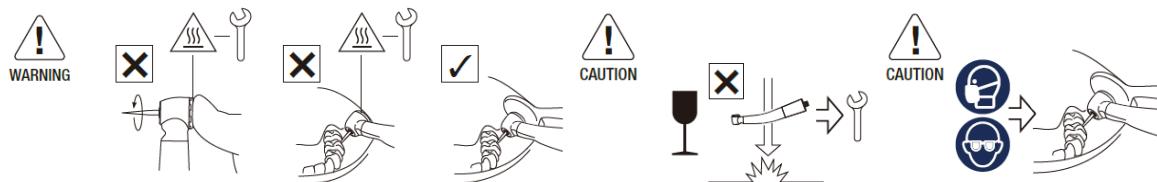
Medical Device

**WARNING:** Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.




**CAUTION** Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

Depressing the Push Button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue.

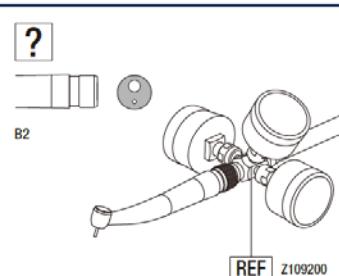
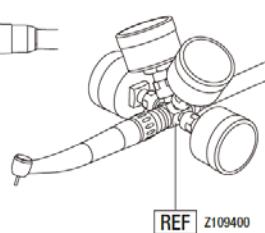
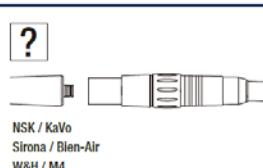
- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not disassemble or alter the handpiece and cartridge, etc. except as recommended by NSK in this Operation Manual.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- The product is delivered in a non-sterile condition. Be sure to sterilize the product and the bur prior to use.
- Perform regular function and maintenance checks.
- Avoid continual eye contact with the LED light. (Z800BL / Z800WL / Z900BL / Z900WL / X450BLED / X450WLED / M800BL / M500BLED / M800WL / M500WLED / M900BL / M600BLED / M900WL / M600WLED / S-Max pico BLED / S-Max pico WLED)
- This handpiece is a LED product classified in the "Exempt Group" (no photobiological hazard) in accordance with IEC62471/EN62471.
- Use a power source which meets the following requirements.
  - The electricity supply of the power source is below 15W both under normal and single-failure conditions.
  - The power source uses a SELV circuit for electricity supply.
  - The output voltage of the power source is within the range recommended by the manufacturer of this product.
- Contact your Authorized NSK Dealer if repairs are necessary.
- An accidental symptom, such as subcutaneous emphysema may be developed in the use of this product for extraction of wisdom teeth. Do not direct air exhausted from the handpiece to the bleeding area (wound) or into periodontal pockets.
- Report all serious events related to the product immediately to the manufacturer and the responsible national authority.

**WARNING** Follow the specifications described in this operation manual as to supply pressure, the bur to be used, etc.

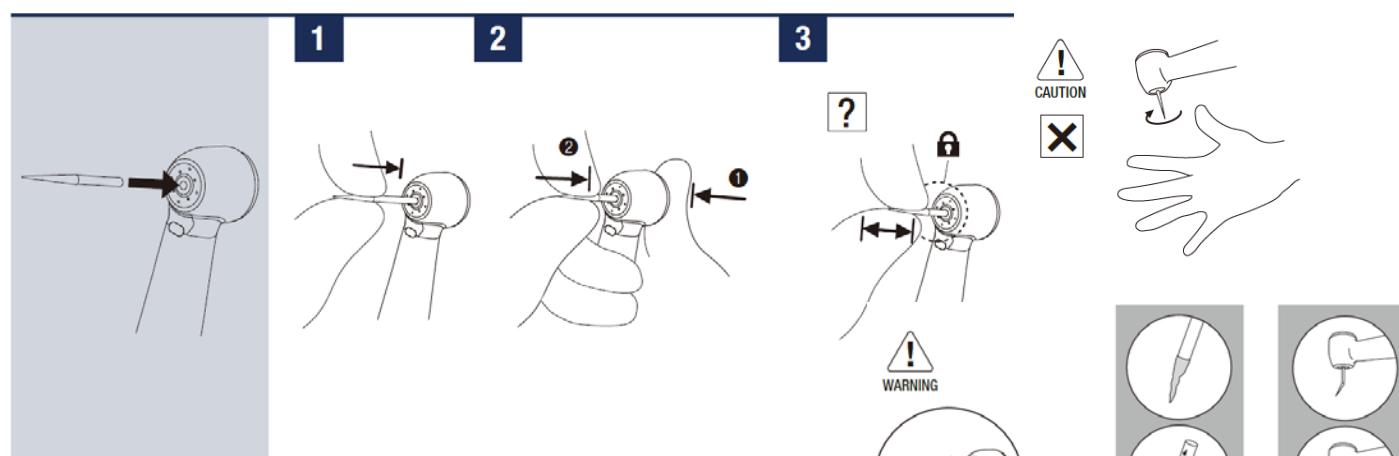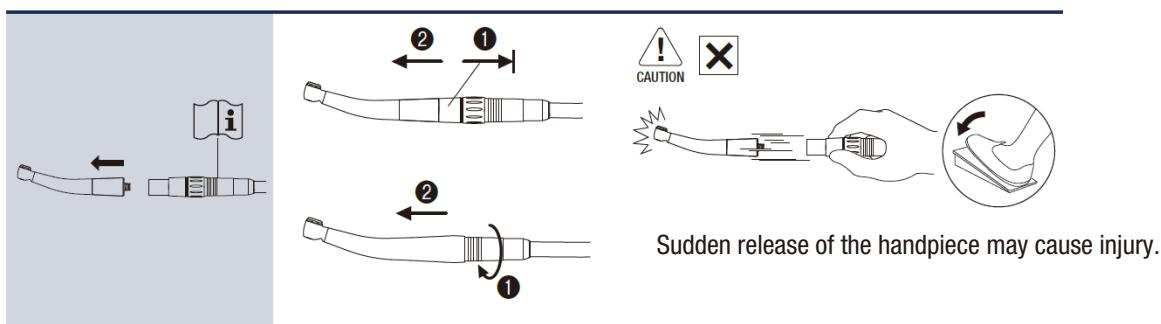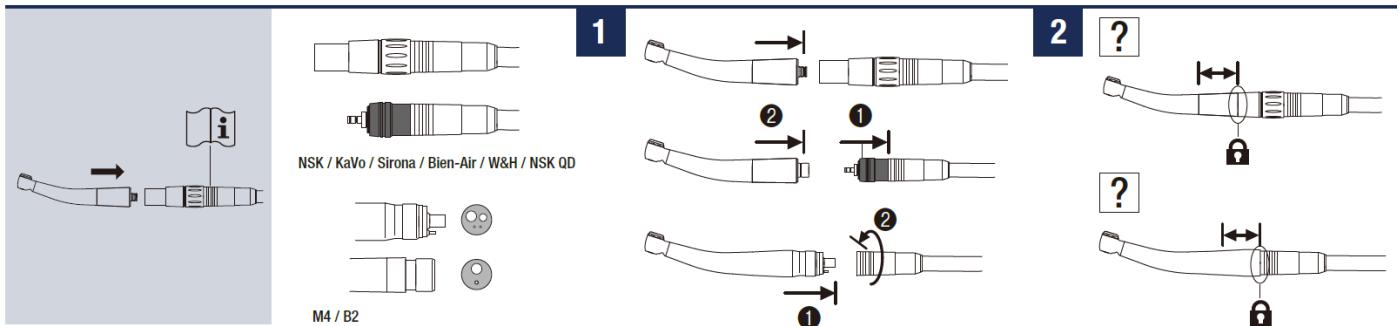**CAUTION** Do not use air contaminated by dust, moisture or oil.

**WARNING** If blood or debris is stuck, wipe in and around the bur insertion hole with a cloth moistened with disinfectant solution until all residues are thoroughly removed. Residual blood or debris may enter the handpiece while changing burs, which could lead to higher risk of cross-contamination.

**CAUTION** Always follow the instructions provided by the bur manufacturer regarding the bur usage and rotation speed, etc.

**CAUTION** Do not apply excess pressure to the bur as it may break or bend and become difficult to remove.



1 | 2

1 | 2 | 3

Checking the Handpiece Before Each  
Use

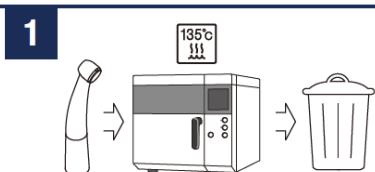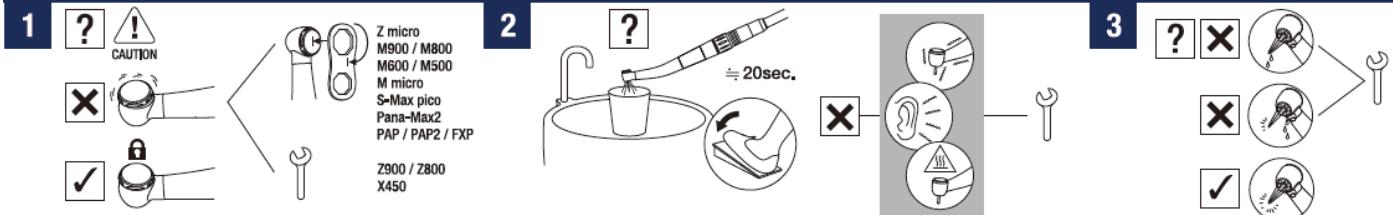Periodical Maintenance Checks  
Every 3 months

Be sure to sterilize before disposing. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose the product for you.

EN Warranty	SK Záruka
DE Garantie	HU Jótállás
FR Garantie	SL Garancija
ES Garantía	HR Jamstvo
IT Garanzia	SV Garanti
PT Garantia	DA Garanti
RU Гарантия	ZH 保固
RO Garanție	NL Garantie
CS Záruka	PL Gwarancja
BG Гаранция	TR Garanti
EL Εγγύηση	FI Takuu

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

**WARNING** To ensure cleaning and sterilization efficacy, use only the following procedures for reprocessing.

**WARNING** Follow local rules, regulations, and guidelines regarding the reprocessing of devices.

**WARNING** Immediately after a treatment (within 1 hour), perform maintenance and then store the handpiece. Failure to properly maintain the handpiece may cause infection, product failure, overheating leading to burn injuries.

**CAUTION** Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorinecontaining solutions, solvents such as benzine or thinner.

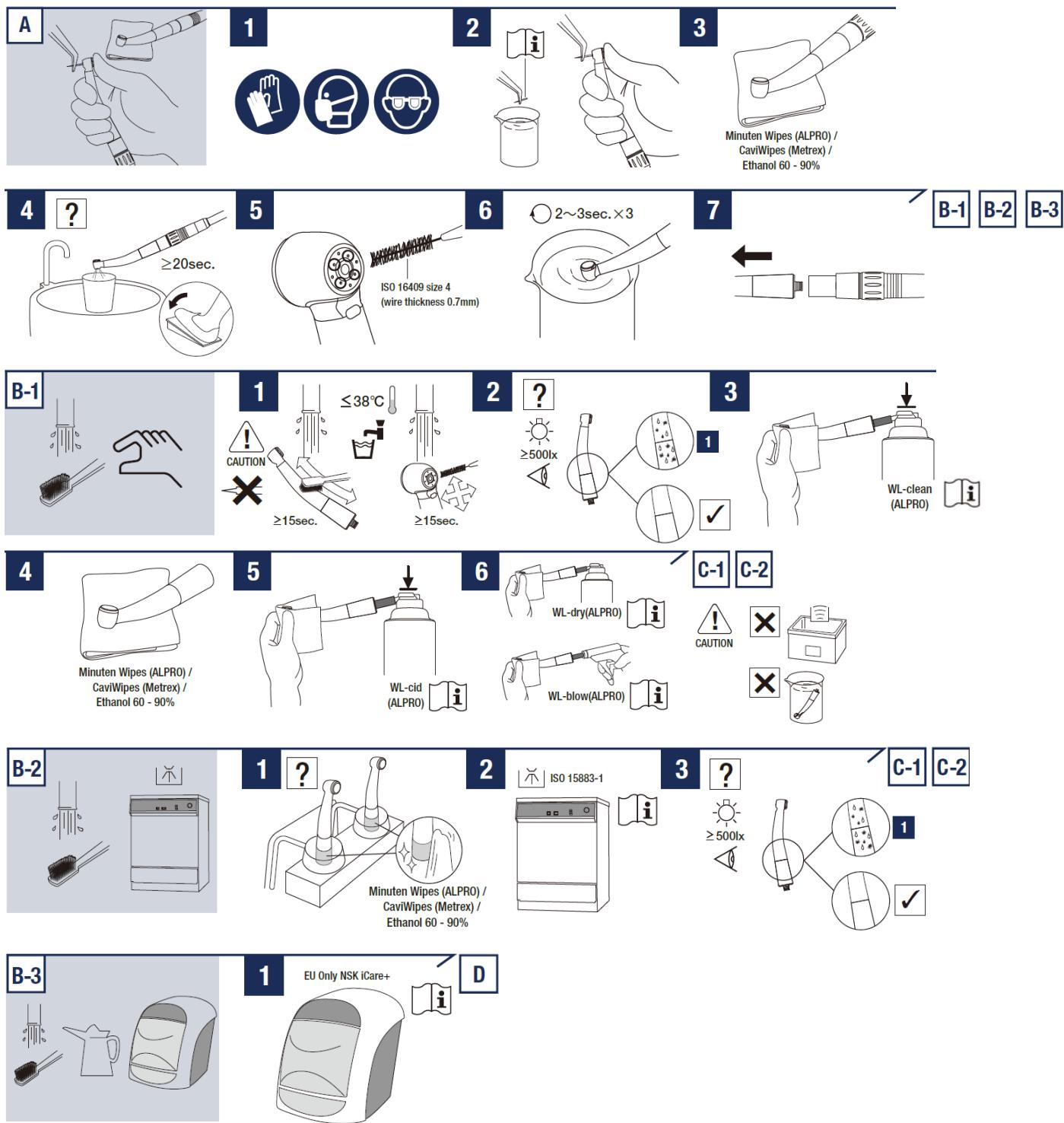
**WARNING** If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components and may lead to blood coagulation inside the handpiece. Coagulated blood may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately lubricate those handpieces with PANA SPRAY Plus.







**CAUTION** Store the product in a well ventilated place out of direct sunlight and within the range of temperature, humidity and pressure

specified in specifications.

**CAUTION** Remove the handpiece from the washer-disinfector immediately after the cleaning, disinfecting, and drying cycle is complete (within 1 hour), otherwise it may cause corrosion.

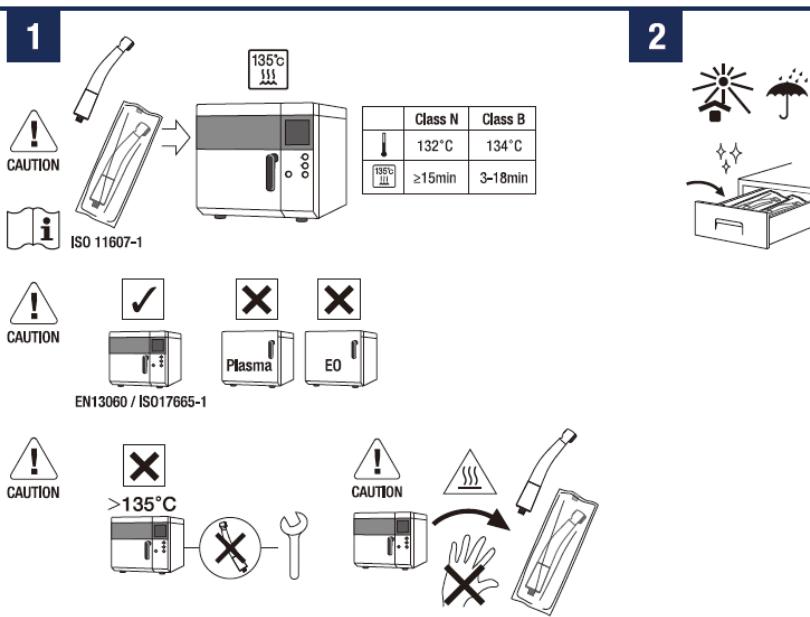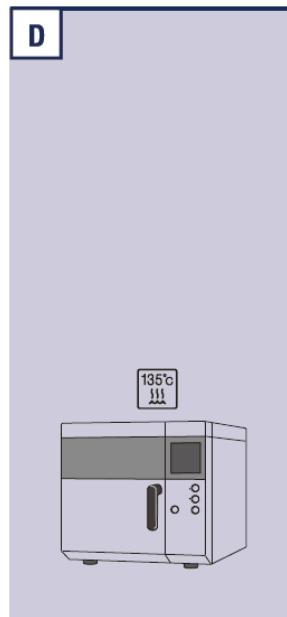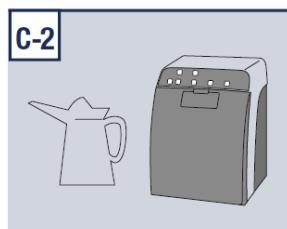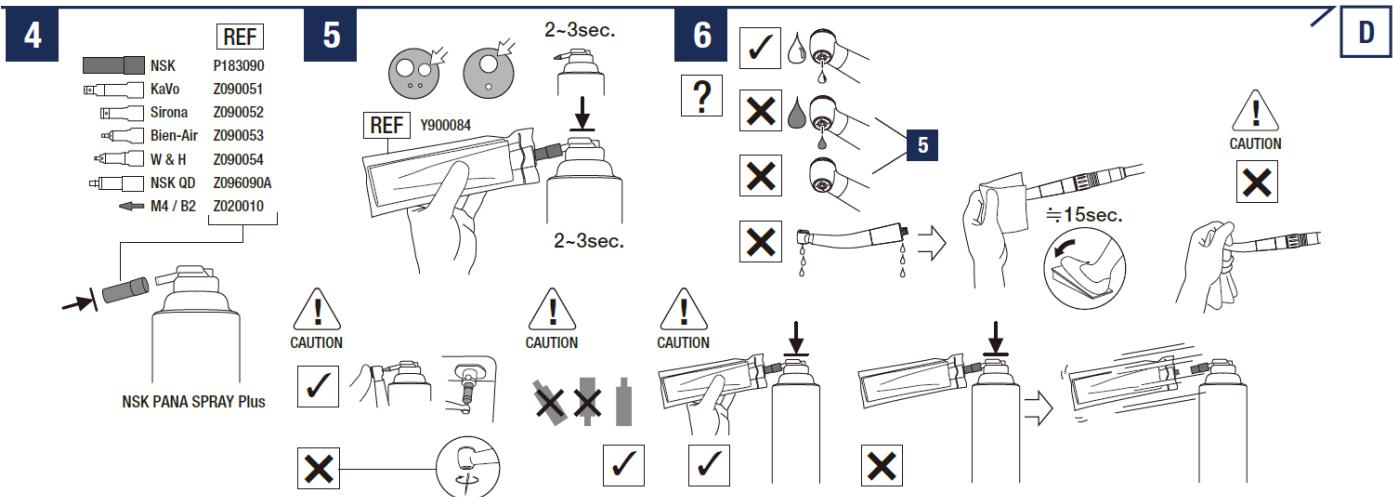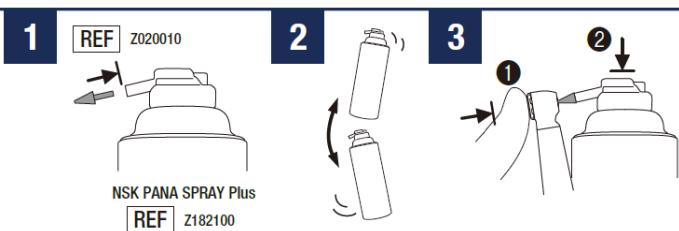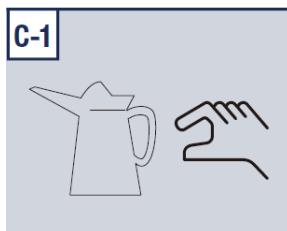**CAUTION** After washing with washer-disinfector, prior to lubrication, dry the product until all internal moisture is completely removed. Remaining moisture inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

**CAUTION** It is verified that this product can be sterilized at least 250 times.

**CAUTION** Clean and lubricate the handpiece prior to sterilization. If blood remains on the internal surface it may become clotted and cause product failure.

**CAUTION** Immediately after sterilization is complete (within 1 hour), remove the handpiece from the sterilizer. Failure to do so may cause corrosion.

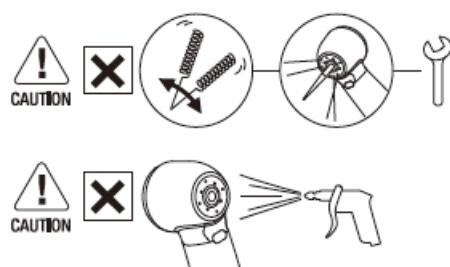
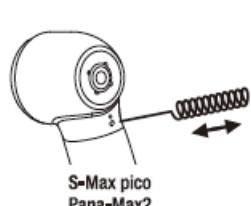
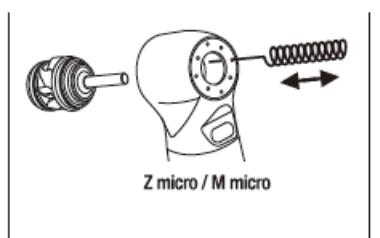
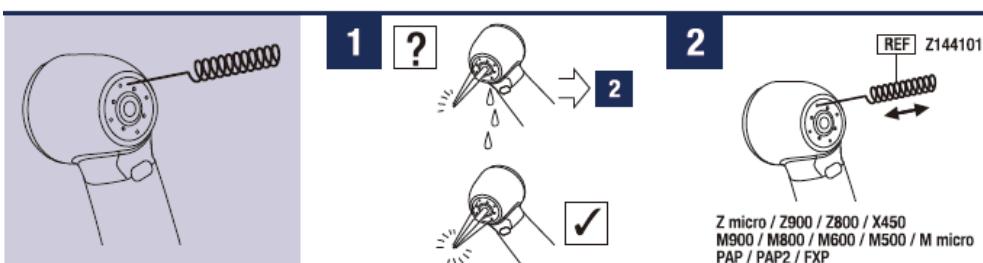
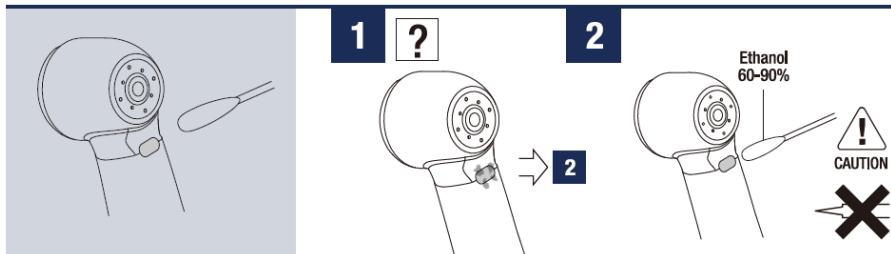
**CAUTION** Do not perform steam sterilization on the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.

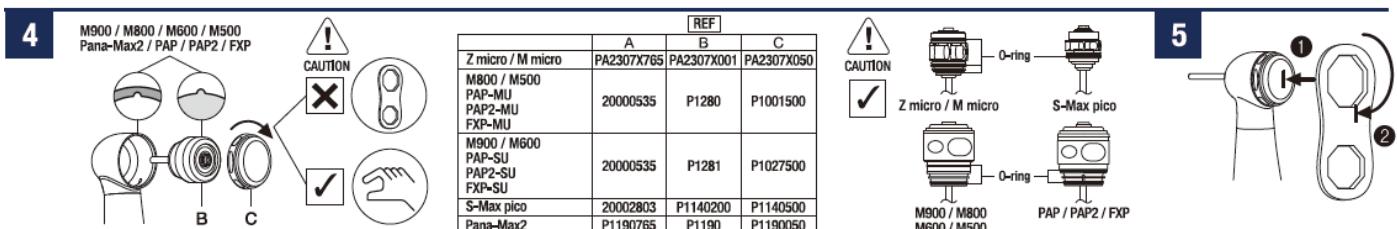
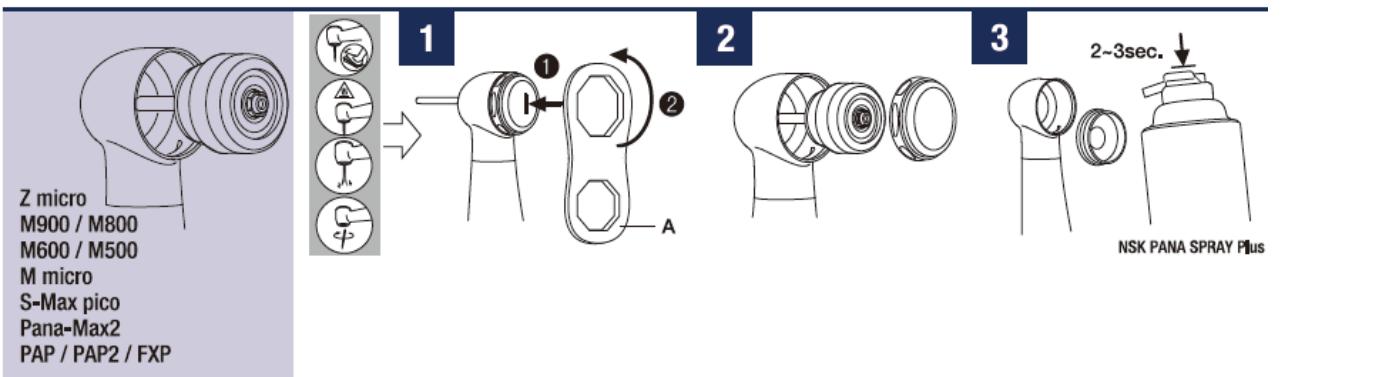
**CAUTION** Sterilization is not guaranteed after the sterilization retention period specified by the manufacturer and seller of the sterilization pouch has elapsed. If the sterilization retention period has elapsed, perform sterilization again with a new sterilization pouch.


**CAUTION** Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.

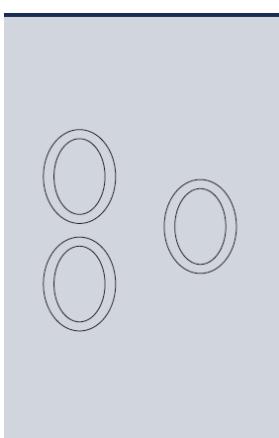
**CAUTION** If another cartridge is used NSK cannot guarantee performance and the handpiece warranty would become invalid.







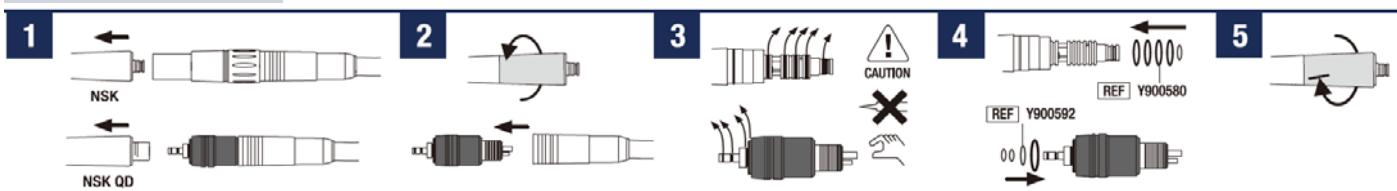
Replace the O-rings if water is present in the exhaust air line. This is an indication of possible water leakage within the coupling. **ALWAYS** change the complete set of O-rings.

**CAUTION** Do not force the new replacement O-ring with excessive pressure.

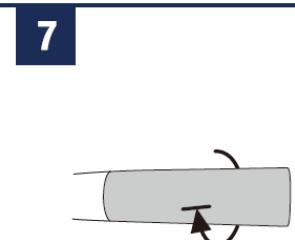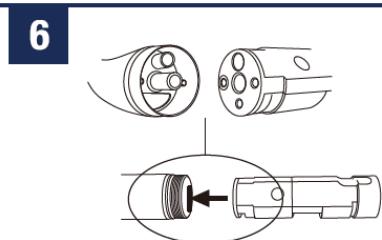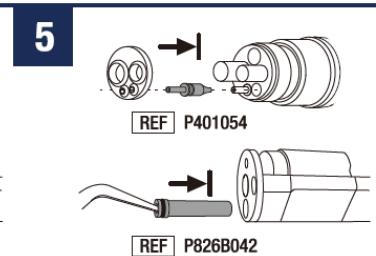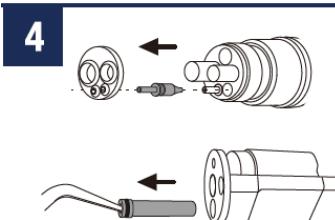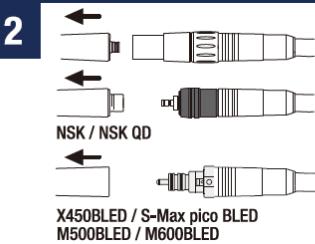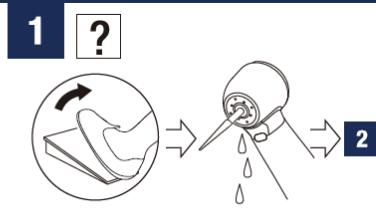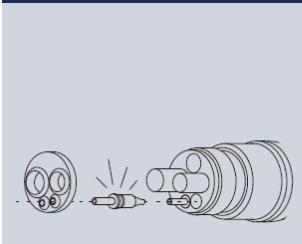**CAUTION** When inserting new O-rings, make sure they are inserted in the correct grooves.

**CAUTION** Make certain that the taper ring is firmly tightened. If the taper ring is loose water and air leakage could occur.

\*Non-CE Model. Not for sale in the EU.

\*Non-CE Model. Not for sale in the EU.

Model									Model				
Z800L									Z800L	✓	✓	✓	
Z800									Z800	✓	✓	✓	
Z800KL									Z800KL	✓	✓	✓	
Z800K									Z800K	✓	✓	✓	
Z800SL									Z800SL	✓	✓	✓	
Z800BL									Z800BL	✓	✓	✓	
Z800WL									Z800WL	✓	✓	✓	
Z800W									Z800W	✓	✓	✓	
Z900L									Z900L	✓	✓	✓	
Z900									Z900	✓	✓	✓	
Z900KL									Z900KL	✓	✓	✓	
Z900K									Z900K	✓	✓	✓	
Z900SL									Z900SL	✓	✓	✓	
Z900BL									Z900BL	✓	✓	✓	
Z900WL									Z900WL	✓	✓	✓	
Z900W									Z900W	✓	✓	✓	
Z micro									Z micro	✓	✓	✓	
Z micro KL									Z micro KL	✓	✓	✓	
Z micro SL									Z micro SL	✓	✓	✓	
X450L									X450L	✓	✓	✓	
X450									X450	✓	✓	✓	
X450KL									X450KL	✓	✓	✓	
X450SL									X450SL	✓	✓	✓	
X450LED									X450LED	✓	✓	✓	
X450WLED									X450WLED	✓	✓	✓	
X450M									X450M	✓	✓	✓	
X450QD									X450QD	✓	✓	✓	
M800L / M500L									M800L / M500L	✓	✓	✓	
M800 / M500									M800 / M500	✓	✓	✓	
M800KL / M500KL									M800KL / M500KL	✓	✓	✓	
M800K* / M500K									M800K* / M500K	✓	✓	✓	
M800SL / M500SL									M800SL / M500SL	✓	✓	✓	
M800WL / M500WLED									M800WL / M500WLED	✓	✓	✓	
M800BL / M500BLED									M800BL / M500BLED	✓	✓	✓	
M900L / M600L									M900L / M600L	✓	✓	✓	
M900 / M600									M900 / M600	✓	✓	✓	
M900KL / M600KL									M900KL / M600KL	✓	✓	✓	
M900SL / M600SL									M900SL / M600SL	✓	✓	✓	
M900K / M600K									M900K / M600K	✓	✓	✓	
M900WL / M600WLED									M900WL / M600WLED	✓	✓	✓	
M900BL / M600BLED									M900BL / M600BLED	✓	✓	✓	
M900W / M600W									M900W / M600W	✓	✓	✓	
M micro									M micro	✓	✓	✓	
M micro KL									M micro KL	✓	✓	✓	
M micro SL									M micro SL	✓	✓	✓	
S-Max pico									S-Max pico	✓	✓	✓	
S-Max pico KL									S-Max pico KL	✓	✓	✓	
S-Max pico SL									S-Max pico SL	✓	✓	✓	
S-Max pico BLEO									S-Max pico BLEO	✓	✓	✓	
S-Max pico WLED									S-Max pico WLED	✓	✓	✓	
S-Max pico M4									S-Max pico M4	✓	✓	✓	
S-Max pico B2									S-Max pico B2	✓	✓	✓	
S-Max pico QD									S-Max pico QD	✓	✓	✓	
Pana-Max2 M4									Pana-Max2 M4	✓	✓	✓	
Pana-Max2 B2									Pana-Max2 B2				
Pana-Max2 QD									Pana-Max2 QD	✓	✓	✓	
Pana-Max2 PTL									Pana-Max2 PTL	✓	✓	✓	
Pana-Max2 KV									Pana-Max2 KV	✓	✓	✓	
PAP-MU M4 / PAP2-MU M4 / FXP-MU M4	$\geq 1.5\text{NL/min (0.20MPa)}$								PAP-MU M4 / PAP2-MU M4 / FXP-MU M4	✓	✓	✓	
PAP-MU B2 / PAP2-MU B2 / FXP-MU B2	-								PAP-MU B2 / PAP2-MU B2 / FXP-MU B2	✓	✓	✓	
PAP-QD MU / PAP2-MU QD / FXP-MU QD	$\geq 1.5\text{NL/min (0.20MPa)}$								PAP-QD MU / PAP2-MU QD / FXP-MU QD	✓	✓	✓	
PAP-SU M4 / PAP2-SU M4 / FXP-SU M4	$\geq 1.5\text{NL/min (0.20MPa)}$								PAP-SU M4 / PAP2-SU M4 / FXP-SU M4	✓	✓	✓	
PAP-SU B2 / PAP2-SU B2 / FXP-SU B2	-								PAP-SU B2 / PAP2-SU B2 / FXP-SU B2	✓	✓	✓	
PAP-QD SU / PAP2-SU QD / FXP-SU QD	$\geq 1.5\text{NL/min (0.20MPa)}$								PAP-QD SU / PAP2-SU QD / FXP-SU QD	✓	✓	✓	

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Model										
Z800L	✓	✓								
Z800	✓	✓								
Z800KL	✓		✓							
Z800K	✓		✓							
Z800SL	✓			✓						
Z800BL	✓				✓					
Z800WL	✓					✓				
Z800W	✓					✓				
Z900L	✓		✓							
Z900	✓		✓							
Z900KL	✓		✓							
Z900K	✓		✓							
Z900SL	✓			✓						
Z900BL	✓				✓					
Z900WL	✓					✓				
Z900W	✓					✓				
Z micro	✓	✓								
Z micro KL	✓	✓								
Z micro SL	✓		✓							
X450L	✓	✓								
X450	✓		✓							
X450KL	✓		✓							
X450SL	✓			✓						
X450LED	✓				✓					
X450WLED	✓					✓				
X450M4							✓			
X450QD								✓		
M800L / M500L	✓	✓								
M800 / M500	✓		✓							
M800KL / M500KL	✓		✓							
M800K* / M500K			✓							
M800SL / M500SL	✓			✓						
M800WL / M500WLED	✓				✓					
M800BL / M500BLED	✓					✓				
M800L / M500L	✓	✓								
M800 / M600	✓		✓							
M800KL / M600KL	✓		✓							
M800SL / M600SL	✓			✓						
M800K / M600			✓							
M800WL / M600WLED	✓				✓					
M800BL / M600BLED	✓					✓				
M800W / M600W						✓				
M micro	✓	✓								
M micro KL	✓	✓								
M micro SL	✓		✓							
S-Max pico	✓	✓								
S-Max pico KL	✓		✓							
S-Max pico SL	✓			✓						
S-Max pico BLED	✓				✓					
S-Max pico WLED	✓					✓				
S-Max pico M4						✓				
S-Max pico B2						✓				
S-Max pico QD							✓			
Pana-Max2 M4										
Pana-Max2 B2							✓			
Pana-Max2 QD								✓		
Pana-Max2 PTL									✓	
Pana-Max2 KV										✓
PAP-MU M4 / PAP2-MU M4 / FXP-MU M4						✓				
PAP-MU B2 / PAP2-MU B2 / FXP-MU B2						✓				
PAP-QD MU / PAP2-MU QD / FXP-MU QD							✓			
PAP-SU M4 / PAP2-SU M4 / FXP-SU M4						✓				
PAP-SU B2 / PAP2-SU B2 / FXP-SU B2						✓				
PAP-QD SU / PAP2-SU QD / FXP-SU QD							✓			
PAP-□□□: 340,000-410,000min <sup>1</sup>									PAP2 / FXP-□□□:	
SN > CBK30001 (M4 / QD)									325,000-410,000min <sup>1</sup>	
SN > CBK30001 (B2)									ISO 1797 Type3 #1.59 - 1.60mm Standard Bur	10.5mm

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